TAKE HOME WHITENING KIT

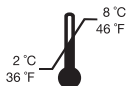
MINT FLAVOR

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Made in USA
1001190.01 120213

PRECAUTIONS

1. Do not use on patients with known sensitivities to peroxide products.
2. Not recommended for pregnant or lactating women.
3. Store bleaching gel out of heat/sunlight. Do not freeze. Store in refrigerator.
4. Dental fillings and crowns do not respond to tooth bleaching.
5. During whitening period: Acidic foods and juices may cause sensitivity. Avoid eating foods that may cause stains for example ketchup, curry, carrots etc.
6. Do not use tobacco products while bleaching.
7. Keep out of the reach of children.

BLEACHING OF TEETH

1. Brush teeth. Load ¼ syringe of gel inside the tray. Gel should be on front surface of teeth when tray is placed in mouth.
2. Place tray on arch and gently press to move gel over the front surface of teeth.
3. Remove excess gel with clean finger, tissue, or soft toothbrush. Gently rinse with water, but do not swallow.
4. When finished, clean teeth and tray with soft tooth brush and cool water. Do not swallow whitening gel. Save cleaned tray for next teeth whitening session.
5. Continue bleaching until wanted result has been achieved.
6. Temporary hypersensitivity can occur. Contact your dental professional for more info.

RECOMMENDED BLEACHING TIMES

Orbis WHITE Take home whitening gel 16% carbamide peroxide

Night bleaching: app. 7–8 hours every night depending on result for 3–8 days

Day bleaching: 2–4 hours depending on result for 3–8 days
16% carbamide peroxide gel will be inactive within 4–5 hours

Orbis WHITE take home whitening gel 6% hydrogen peroxide

Day bleaching: 30–60 min depending on result for 5–10 days

Not to be used on a person under 18 years of age. To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.

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